


Top view of scanner bed, with full sheet of film positioned at the center

**Epson1000XL** scanner

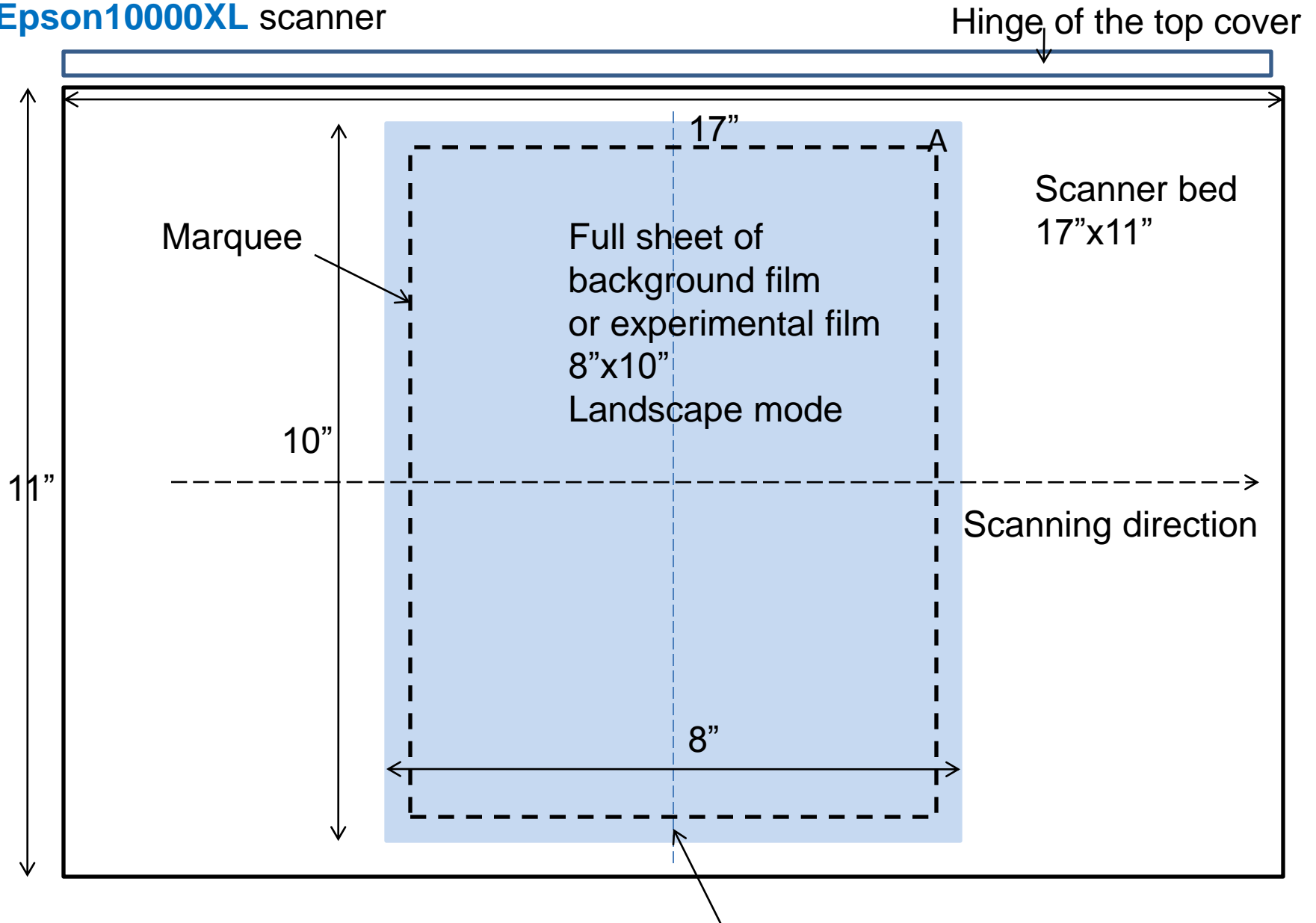

Hinge of the top cover

Marquee

17"

Scanner bed  
17"x11"

Full sheet of  
background film  
or experimental film  
8"x10"  
Landscape mode

10"

Scanning direction

8"

11"

Figure 1

Center reference line of the scanner bed

Top view of scanner bed, with calibration films and 0 cGy films positioned at the center  
**Epson 10000XL** scanner

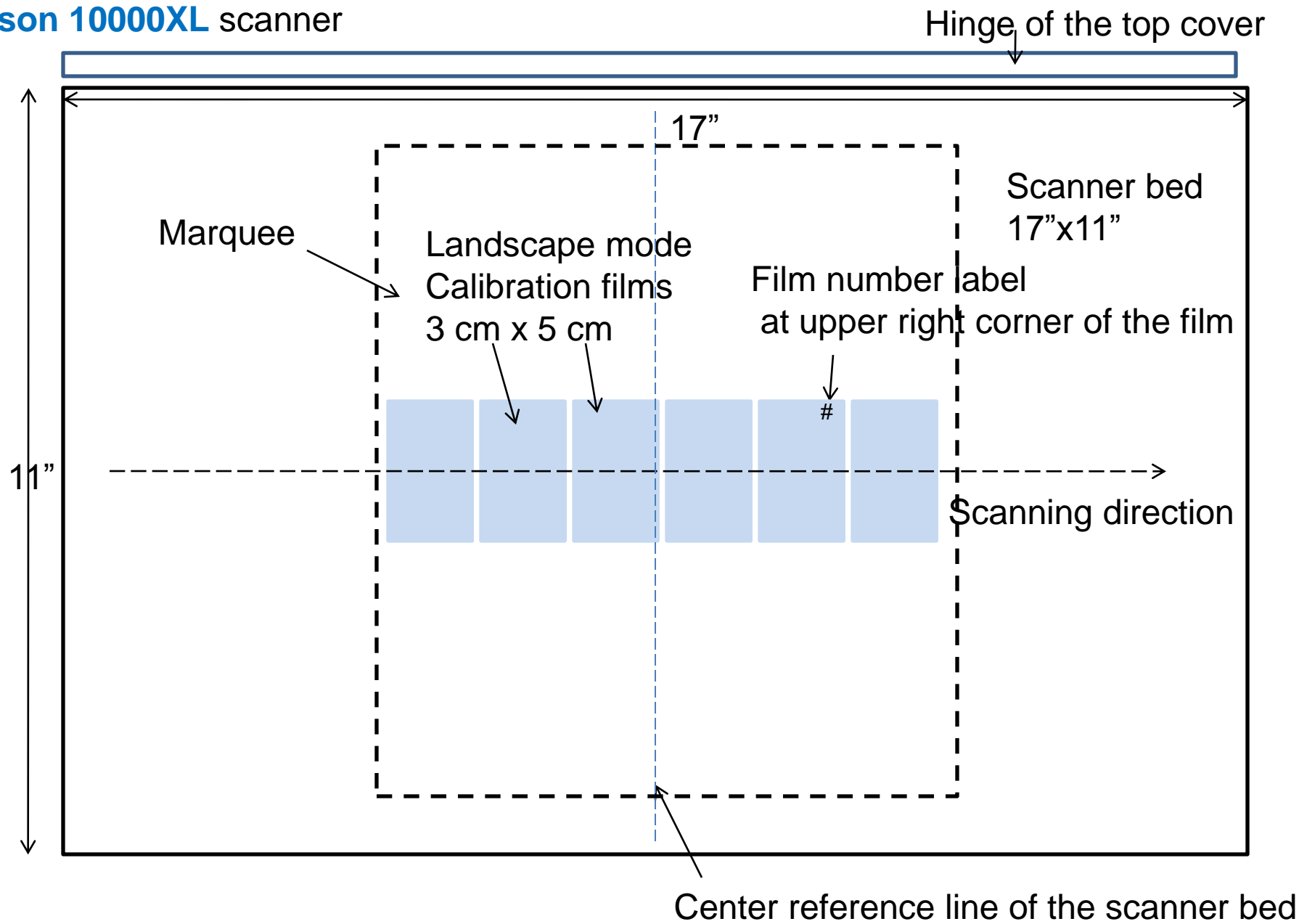


Figure 2

Top view of scanner bed, with full sheet of film positioned at the center

**Epson V750** scanner

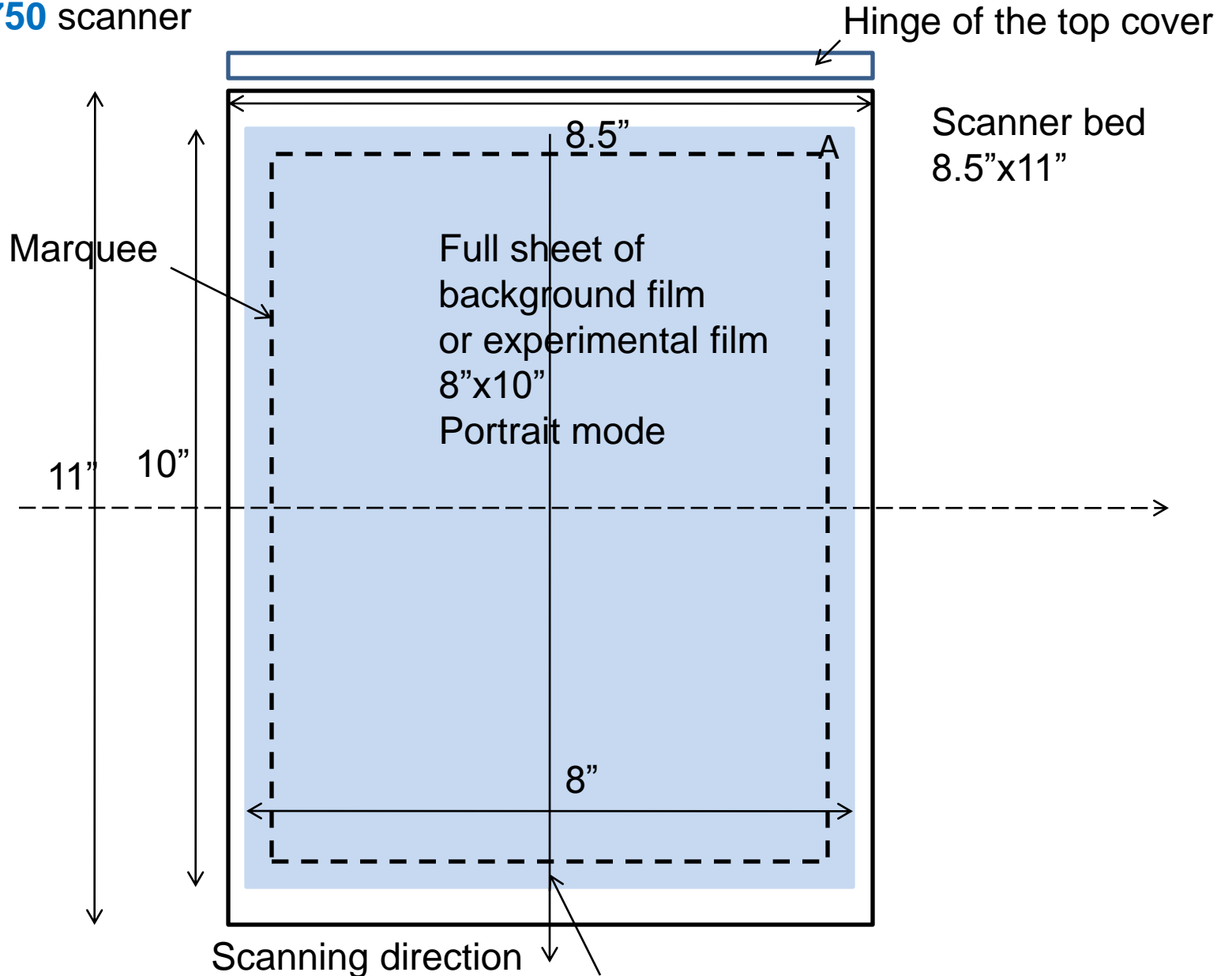


Figure 3

Center reference line of the scanner bed

Top view of scanner bed, with a calibration film positioned at the center

**Epson V750** scanner


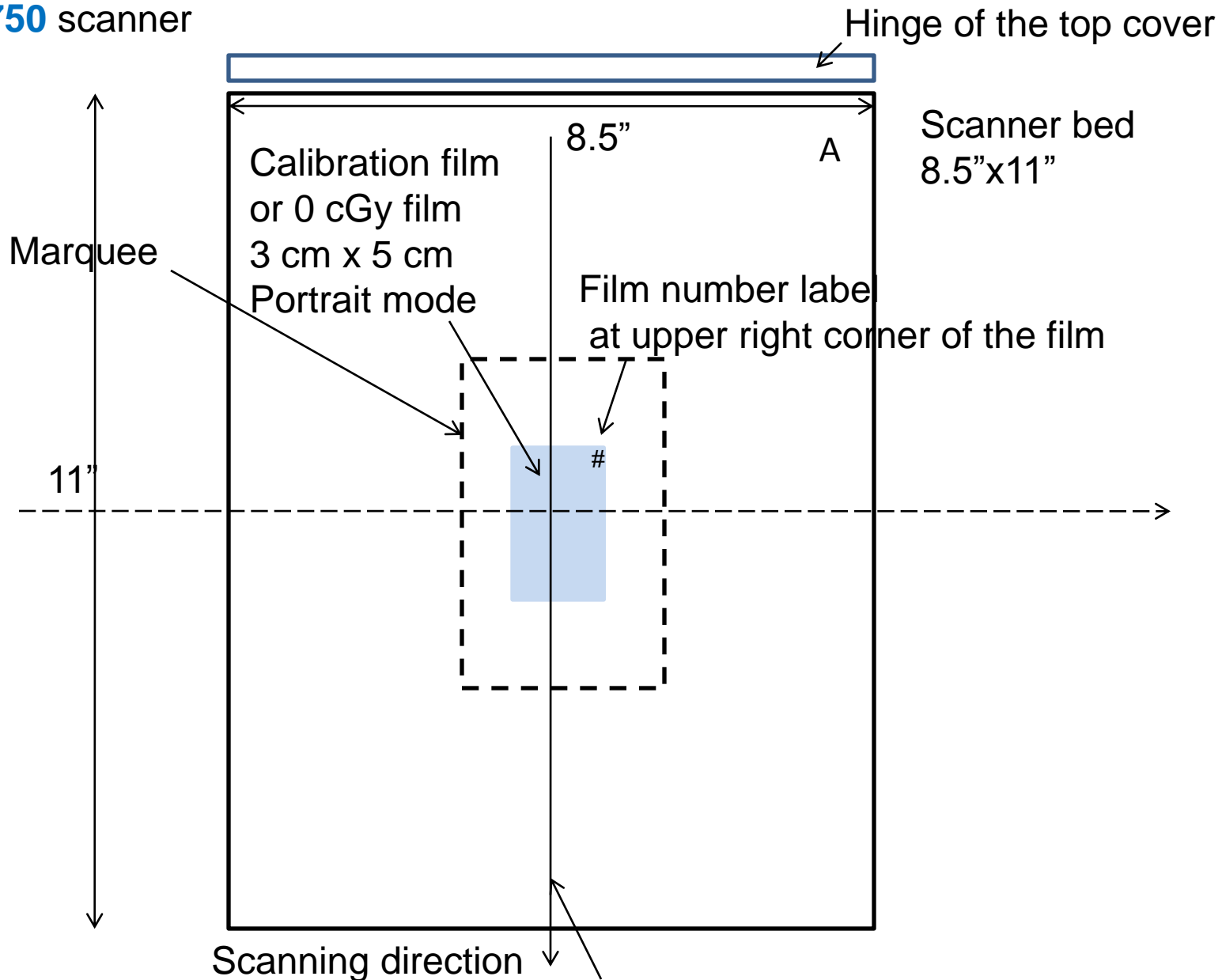


Figure 4

Center reference line of the scanner bed

Top view of scanner bed, with full sheet of film positioned at the center

**Epson V750** scanner

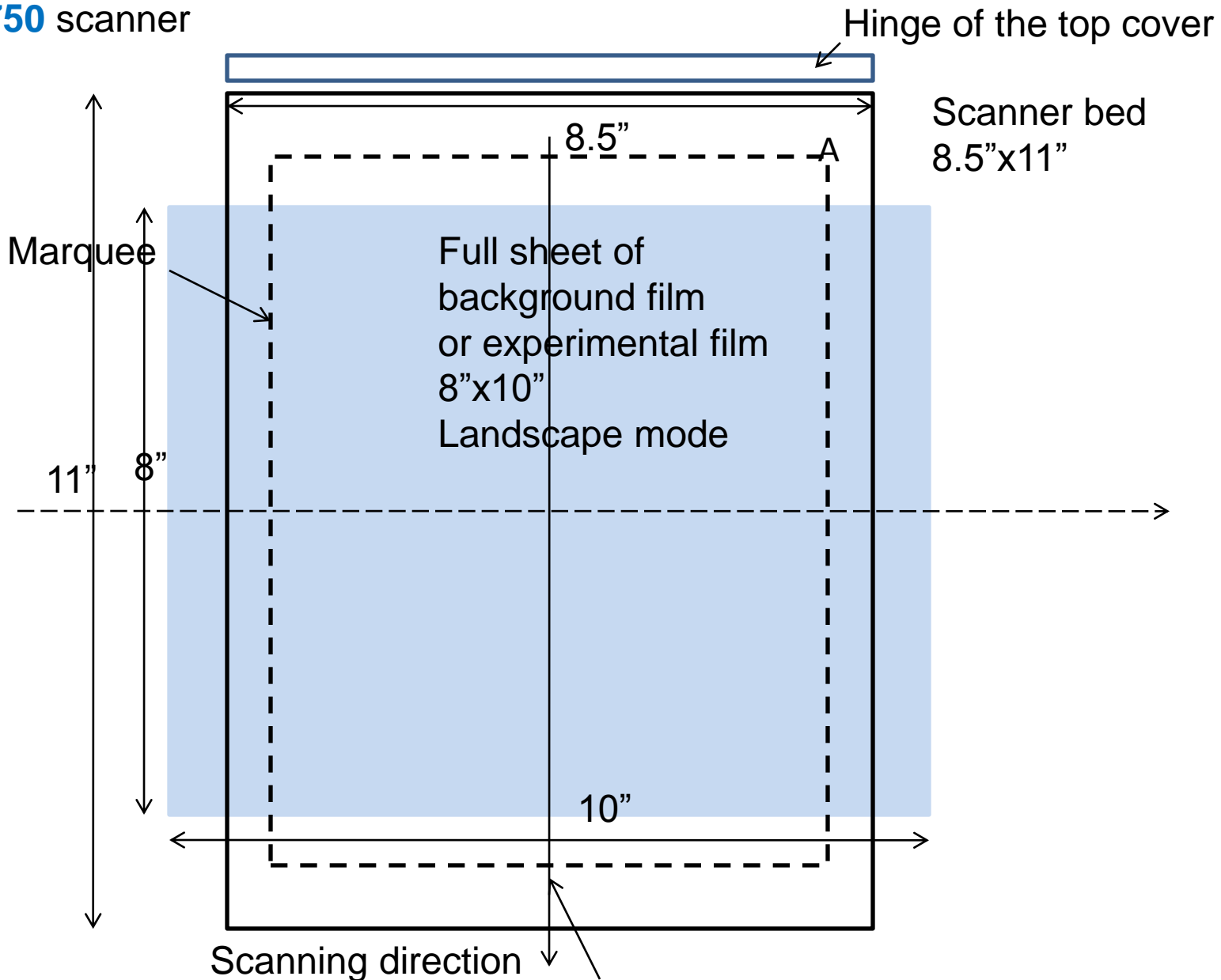

Scanner bed  
8.5"x11"

Full sheet of  
background film  
or experimental film  
8"x10"  
Landscape mode

Marquee

Hinge of the top cover

8.5"

11"

8"

10"

Scanning direction

Center reference line of the scanner bed

Figure 5

**Impossible to scan the entire film sheet in Landscape mode on V750 scanner**

Top view of scanner bed, with a calibration film positioned at the center

**Epson V750** scanner

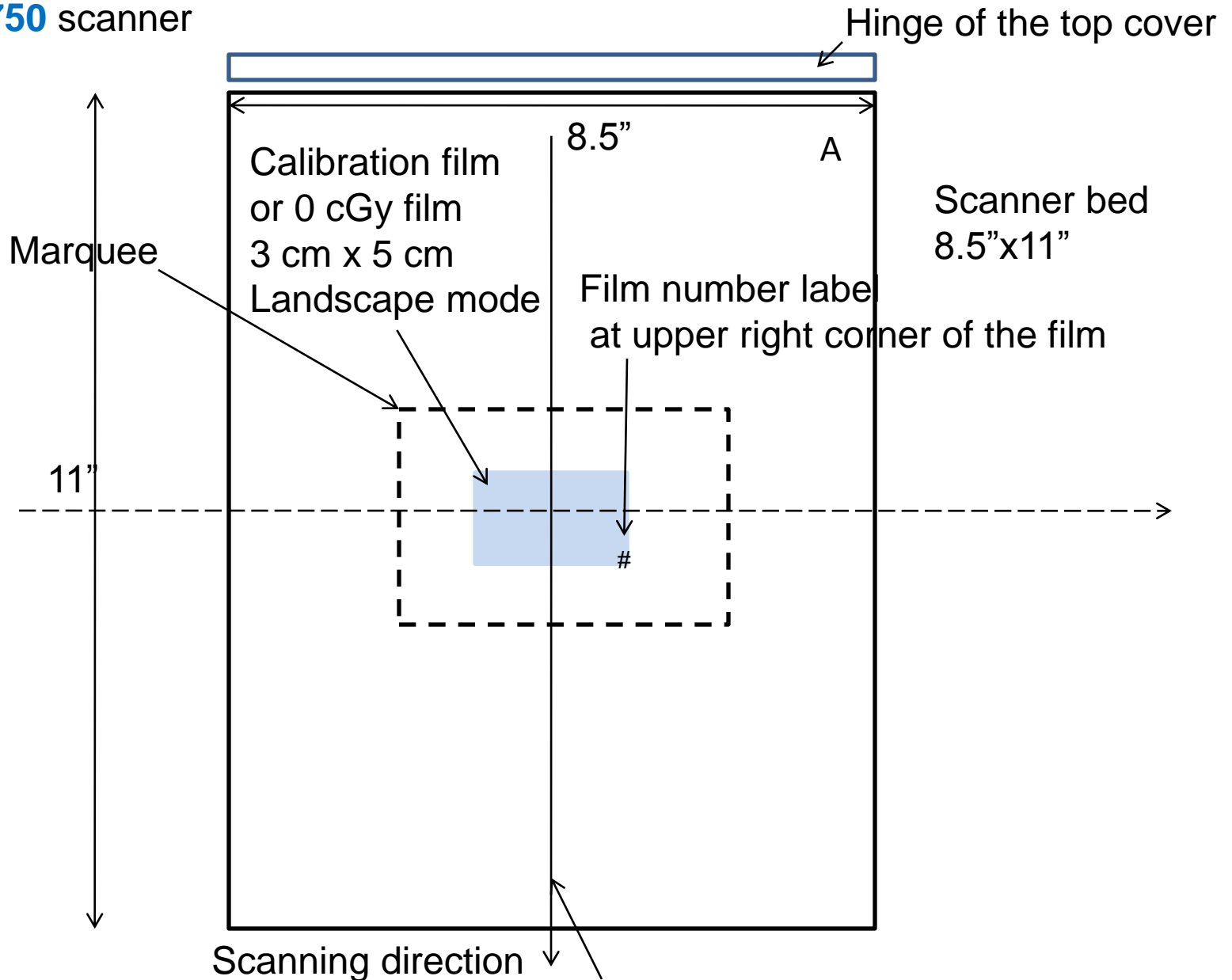


Figure 6

Center reference line of the scanner bed